

RIGID NIGHT WRIST BRACE

CARP'IMMO™

CARPAL TUNNEL SYNDROME

PROVIDES MAXIMUM SUPPORT & COMFORT FOR WRIST



ADJUSTABLE SELF-GRIPPING STRAP

> For a
customisable,
secure fit.

SOFT, ULTRA-THIN FABRIC

> To ensure
optimal
comfort



REINFORCED LINING

> Wrist held
in a neutral,
resting position.

INDICATIONS:

Regular use of the CARP'IMMO™ RIGID NIGHT WRIST BRACE (at night or at rest) is recommended for carpal tunnel syndrome, musculoskeletal disorders, swelling or wrist tendonitis.

CONTRA-INDICATIONS:

Use only on clean, intact skin.

PRECAUTIONS FOR USE:

In case of severe oedema: monitor the hand during use. Remove the device if excessively tight and hampering circulation. If in doubt, ask your doctor's or pharmacist's advice.

Do not apply any cream or ointment before putting on the orthosis. Do not use near a flame or incandescent material.

SPECIFIC FEATURES:

The ultra-light EPITACT® brace adapts perfectly to your wrist and comfortably holds it in a neutral position, thanks to the integrated semi-rigid lining.

DESIGN: The CARP'IMMO™ brace adapts perfectly to the shape of your hand due to the design of the integrated semi-rigid lining. It's so thin and lightweight you won't even know you're wearing it! The brace opens easily to accommodate your hand and the strap allows you to adjust it perfectly to your wrist.

TIP:

To supplement the rigid wrist brace, we recommend that you wear the EPITACT® CARPACTIV™ flexible wrist brace in all situations in which you need to use your hand.

LIFESPAN:

Hand washing at 30°C / 86 F, with a mild soap, without detergent or bleach. Rinse well. Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers.

COMPOSITION: 59% fabric (polyamide, Elastane, polyurethane) 41% thermoplastic elastomer

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2018.



Hand washable.



Do not iron
or steam.



Do not dry clean.



Do not bleach.



Do not
tumble-dry.



See packaging.

Product of MILLET Innovation research.



MILLET Innovation ZA Champgrand
26270 Lorio/Drôme - FRANCE

MILLET
INNOVATION

www.epitact.com

N19497-A
09/2020

