

DIGITUBES® with EPITHELIUM™

INDICATIONS

Digitubes® protect, relieve pain and eliminate corns.

CONTRAINDICATIONS

Do not use on injured skin. In case of arteritis, diabetes, lower limb neuropathy or ulcers, seek advice from your doctor or podiatrist before use and then regularly monitor your foot.

TIPS FOR USE

- Use a good pair of scissors to cut the Digitube to the required length.
- Remove the protective film covering the gel.
- Turn the Digitube® inside out and place it on the toe, making sure that the Epithelium™ gel is in contact with the area you wish to protect.

FEATURES

Excessive rubbing and pressure cause hard skin called corn. The presence of EPITHELIUM™ gel in Digitubes® protects toes, relieves pain immediately and helps corns to disappear.

DESIGN

Digitubes® are very slim patented protectors with built-in 1mm EPITHELIUM™ gel only in the semi circumference so they can be worn on adjacent toes with no discomfort.

CLEANING & LIFESPAN

Hand washable and reusable. Using 2.5cm of Digitube, you can protect a corn for an average of 10 days.

COMPOSITION: 73% silicone, 27% polyamide/elastane fabric.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.



Hand washable.



Do not iron or steam.



Do not dry clean.



Do not bleach.



Do not tumble-dry.



See packaging.

Product of MILLET Innovation research.



MILLET Innovation ZA Champgrand
26270 Loriol/Drôme - FRANCE



MILLET
INNOVATION

www.epitact.co.uk