

FLEXIBLE WRIST BRACE

CARP'ACTIV™

CARPAL TUNNEL SYNDROME

PROVIDES PROTECTION & COMFORT FOR WRIST



INDICATIONS:

Wearing the CARP'ACTIV™ flexible wrist brace on a daily basis is recommended for those suffering from carpal tunnel syndrome, musculoskeletal disorders and wrist tendonitis.

CONTRAINDICATIONS:

Only use on clean, intact skin.

PRECAUTIONS FOR USE:

Not recommended for prolonged computer keyboard use. Do not use near a flame or incandescent material.

SPECIFIC:

This brace has been designed to provide support for your wrist while allowing the full use of your fingers. The CARP'ACTIV™ stabilises the wrist in a neutral position and limits excessive hand and wrist movements.

DESIGN:

Thanks to its high-tech fabric and patented design (guaranteeing a snug fit), the CARP'ACTIV™ flexible wrist brace provides maximum support and comfort for the wrist.

LIFESPAN:

Machine-washable in laundry net (included in the box). Squeeze to expel moisture. Do not expose while drying to a source of direct heat (radiator, sunlight). Do not use any stain removers.

COMPOSITION:

100% polyamide/elastane fabric.
The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2018.



Machine washable.



Do not iron or steam.



Do not dry clean.



Do not bleach.



Do not tumble-dry.



See packaging.

Product of MILLET Innovation research.



MILLET Innovation ZA Champgrand
26270 Lorio/Drôme - FRANCE

MILLET
INNOVATION

www.epitact.com

N19455EN-A
09/2019